


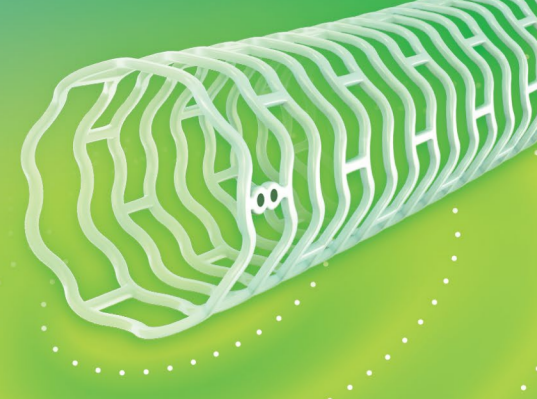

CASE REPORT

Courtesy of J.D. Corl, MD, FACC, FSCAI

SUCCESSFUL TREATMENT OF POSTERIOR TIBIAL CTO IN BILATERAL TMA PATIENT WITH AN ESPRIT™ BTK SCAFFOLD

This case study outlines the treatment and follow-up of a 60-year-old male with history of bilateral TMA and a non-healing ulcer on the plantar surface of right foot with CTO in both AT and PT. This procedure involved the use of a calcium modifying balloon and an Esprit™ BTK scaffold, resulting in successful revascularization.

PATIENT PRESENTATION

A 60-year-old male with a complex medical history, including PAD, CLI, bilateral TMA, DM-2, tobacco abuse, HLD, and HTN, presented with a non-healing wound on his right foot. The patient was categorized as Rutherford Becker Category 5.

Diagnostic Findings

ABI was 0.39 on the right side, with CT imaging revealing moderate non-obstructive plaque in the right CIA. The right Superficial Femoral Artery (SFA) and popliteal artery showed diffuse non-obstructive plaque, and the right AT and PT arteries were completely occluded (**Figure 1A**).

I/O Inflow/Outflow

Pre-treatment flow was normal above the target tibial lesion, but outflow beyond the target lesion was only distal filling via collaterals.

P Prepare the Lesion

To prepare the lesion, a 3.5 x 80 calcium modifying balloon was used, resulting in a focal dissection in the mid-right PT. Angioplasty was performed in the right popliteal artery and right TPT with a 5x60 drug-coated balloon via right PT access, in the right peroneal artery with a 2.5x 40 balloon via left radial access, in the right PTA with a 3.5x150 balloon via right PT access.

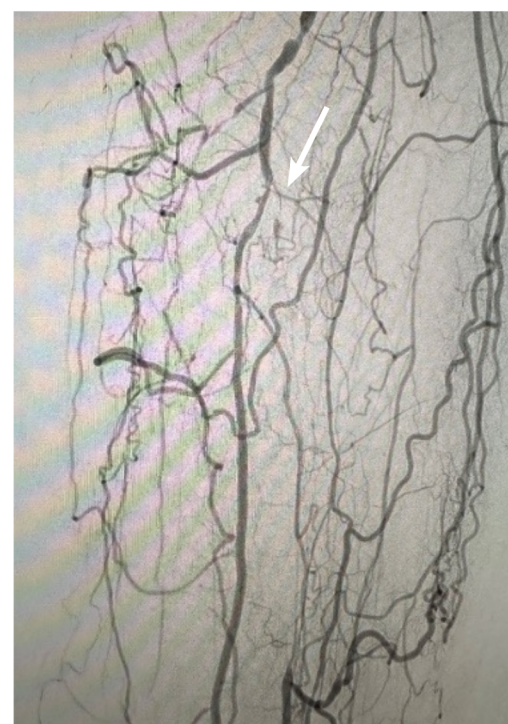


Figure 1A
Pre-treatment angiography

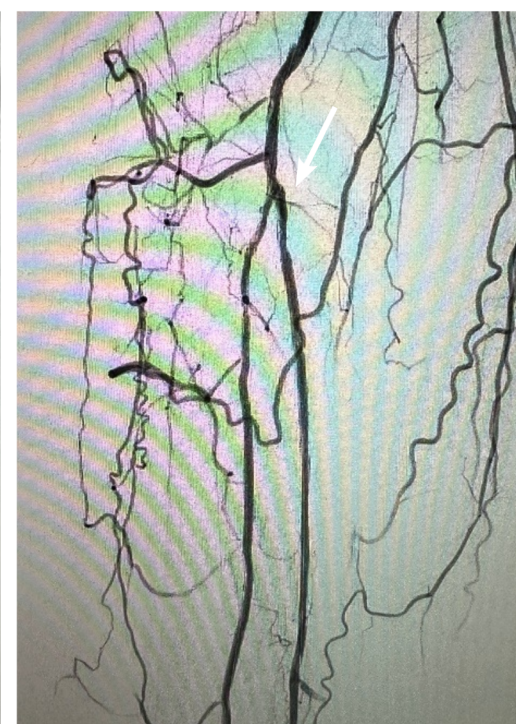


Figure 1B
Post-treatment angiography

S Sizing

The target lesion in the right PT, had a length of 25 mm and a proximal reference vessel diameter of 3.04 mm (**Figure 2**) via IVUS. An 0.014 guidewire was used to deliver a 3x38 mm Esprit BTK scaffold to the proximal/mid-right PT (**Figure 3**).

P Post-Dilatation

Post dilatation of the Esprit™ BTK scaffold was performed with a 3.0 x 40mm semi compliant balloon to ensure apposition.

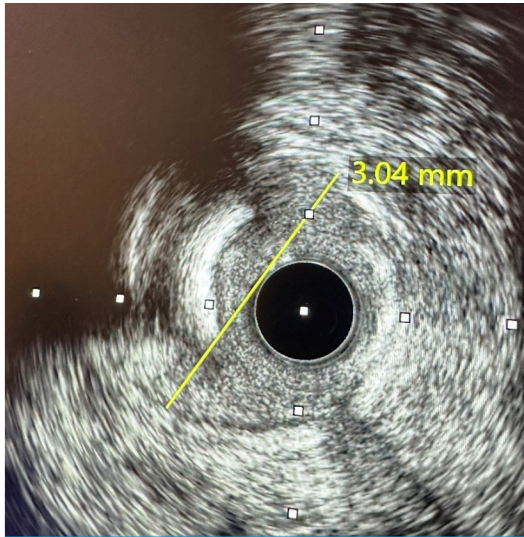


Figure 2
Sizing the vessel with IVUS

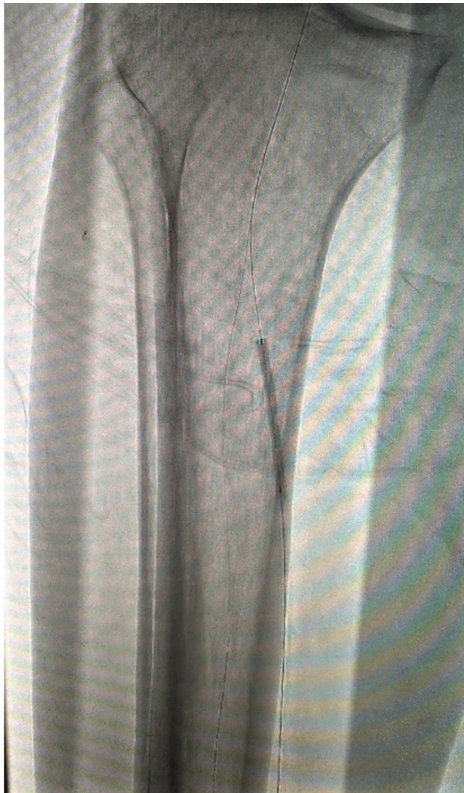


Figure 3
Deploying scaffold

Post-Procedure Outcome

The follow up duplex scan showed patency and brisk flow in the PT with wound healing in 4-5 weeks (**Figure 1B**).

CAUTION: Certain configurations of the devices within may not have been licensed in accordance with Canadian law. Contact your local sales representative for the regulatory status of the device(s) in Canada. This product is intended for use by or under the direction of a physician. Prior to use, reference the Instructions for Use, inside the product carton (when available) or at manuals.eifu.abbott for more detailed information on Indications, Contraindications, Warnings, Precautions and Adverse Events as applicable to Canada. This material is intended for use with healthcare professional only.

Illustrations are artist’s representations only and should not be considered as engineering drawings or photographs. Photos on file at Abbott.

Abbott
6975 Creditview Road Unit 1 Mississauga, Ontario L5N8E9 Canada Tel: 1-866-552-5753

™ Indicates a trademark of the Abbott Group of Companies.

Information contained herein intended for Healthcare Professionals in CANADA ONLY
©2025 Abbott. All Rights Reserved. MAT-2508425 V1.0

